



Step-by-Step Restorative Protocol

■ DELIVERY APPOINTMENT

Delivery of the Final Prosthesis



- Retighten multi-unit abutments to manufacturer-recommended torque.



- Seat the BruxZir® prosthesis on the multi-unit abutments.

■ DELIVERY APPOINTMENT

Delivery of the Final Prosthesis



- Hand-tighten the enclosed final prosthetic screws, alternating from one side to the other.



- Tighten the screws to 15 Ncm. Wait approximately 5 minutes and retorquer screws.



- Place Teflon tape in each screw access channel.



- Fill screw access channels with light cure composite to prevent bacteria build-up.



- Confirm occlusion.



- Make adjustments if necessary.

A BruxZir™ Adjustment & Polishing Kit can be purchased via Glidewell Direct.



- If adjustments were necessary, polish the affected areas. Reinforce oral hygiene instructions.

Have questions? Call 866-861-7955 or check out our how-to video below.



Deliver BruxZir Prosthesis
bit.ly/deliverbruxzirprosthesis

■ POST-DELIVERY CHECK

- Verify occlusion
- Review oral hygiene instructions
- Establish recall schedule

■ MAINTENANCE APPOINTMENTS

How to Maintain the Final Prosthesis

Note: Removal of the prosthesis NOT recommended. If the prosthesis is removed, prosthetic screws MUST be replaced.

- Six-month hygiene appointment
 - Perform prophylaxis under the prosthesis
- Twelve-month (annual) hygiene appointment