



PREPARATION GUIDELINES

ANTERIOR

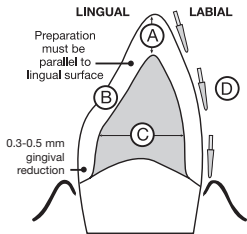


- A. 1 mm incisal reduction
- B. 0.8 mm middle third reduction
- C. Labial and lingual walls must be convergent
- D. Preparation should be cut in three planes

POSTERIOR

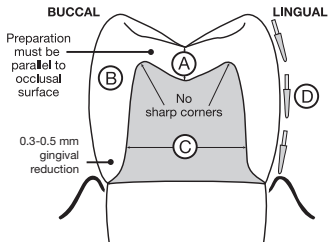


- A. 1 mm occlusal reduction
- B. 0.8 mm middle third reduction
- C. Buccal and lingual walls must be convergent
- D. Preparation should be cut in three planes

VENEERS

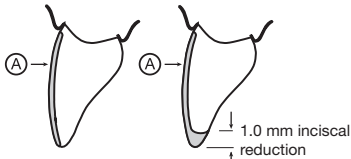

- A. 0.3 to 1.0 mm labial reduction

LABIAL OR 360° MARGIN

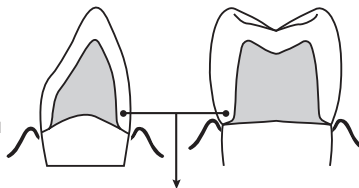

- 0.3-0.5 mm gingival reduction using rounded shoulder margin design