

Producer



After screwing the metal T-nut and black thumbscrew onto the articulator, you are ready to articulate your model work.



Models are easily removed and placed back onto the articulator with just a few twists of the thumbscrew.



The Producer features a rear pin set that improves accessibility for anterior cases.


Dentronics



Dentronics offers an incisal pin adjustment marked in 1mm increments; handy when opening the vertical dimension.



Models are easily removed and placed back onto the articulator with just a few twists of the thumbscrew.

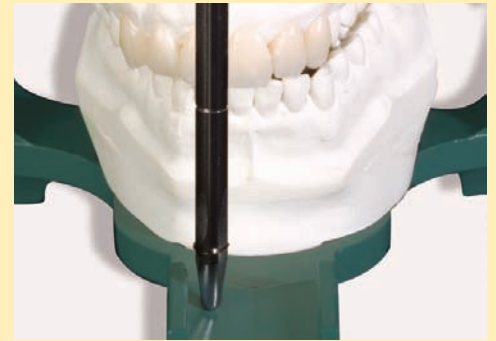

The Dentronics provides protrusive and lateral movements for optimal occlusion of your restoratives.

Disposable



Pour model base into rubber mold. The disposable articulator is compatible with current slotted molds.



After articulating your models, simply glue the disposable articulator pieces to lock into place.



Models can easily go into lateral excursions and back into centric without the hinges popping.